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Performix™ HD X-Ray Tube

For GE Discovery CT7500
D3701T / D3702T / D3703T / D3704T

The Performix HD X-Ray Tube has re-invented the Performix family of GE tubes, invigorating the proven Performix brand with high definition technology. GE's Designed-for-Six-Sigma Performix tube product has been validated by 10 years in operation and over 25,000 installations. Supporting your clinical ability to see more and know more with less dose, the Performix HD satisfies this performance expectation by helping to deliver exceptional image quality.

Dynamic Focal Spot Control Translates Into Image Quality

Through dynamic focal spot control, Performix HD customizes focal spot size to maximize image quality. This technology relies on complex integration between components in the imaging chain to help enable:

- *Consistent Focal Spot Control*, maintaining image quality across images within the acquired dataset
- *High mA for Small Focal Spot*, bringing exceptional image quality to larger patient scans
- *Data Sampling*, offering additional information necessary for true anatomy reconstruction by the CT system
- *Fast kVp Switching*, necessary to support Gemstone Spectral Imaging
- *Outstanding Spatial Resolution*, as part of the image chain—along with the Gemstone Detector and Data Acquisition system—delivering spatial resolution improvements.

Integration

GE's integration of tube, generator and system dose-management technologies provides customers the imaging flexibility and power to meet demanding imaging requirements while optimizing patient care.

Maximize throughput

The Performix HD X-Ray Tube handles protocols while delivering high capability by helping to minimize cooling delays with greater than 5000 W dissipation rate.

DOC0379527 rev 6

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Performix HD Specifications

D3701T/D3702T/D3703T/D3704T

(Tube Model Numbers 5195800 / 5195800-5 / 5195800-6)

Tube Insert Focal Spots

Small Focal Spot (680 max mA)

- 1.0 x 0.7 per IEC 60336/2005

Large Focal Spot (835 max mA)

- 1.6 x 1.2 per IEC 60336/2005

GSI Focal Spot (890 max mA) (GSI Scanning only)

- 2.0 x 1.2 per IEC 60336/2005

Target Angle: 7 degrees

Thermal Ratings:

Efficient anode heat transfer and casing design eliminates inter-patient delays for demanding helical scans

Anode

- Max anode heat content: 5.7 MJ
- Nominal anode input power: 100kW

Housing

- Max x-ray tube assembly heat content: 8MJ
- Max continuous heat dissipation: 5.1 kW

X-Ray Tube Housing Assembly

Anode-Grounded Technology

- Nominal tube voltage: 140kVp
- Leakage technique factor: 140 kV, 32.1 mA
- Quality equivalent filtration: Min 3.91mm Al equiv at 70 kV

Warranty

The published warranty in effect on date of shipment shall apply. Right reserved to make changes.

Regulatory Listings

CE Mark

Intertek (ETL) certified to meet applicable IEC and CSA standards.

This tube unit has been designed to meet or exceed all applicable 21 CFR Subchapter J performance standards for diagnostic X-ray equipment as stipulated by the Radiation Control for Health and Safety Act.

Included License

The GE Performix HD tube includes a standard license that automatically enables the use of tube-dependent advanced applications. The use of a non-Performix HD tube will require an additional license for the activation of these features.

Single Load Rating

Permissible mA values for single 5-second exposure from cold

Scan Mode	kV	Small Focal Spot mA	Large Focal Spot mA
Normal	80kV	620	700
	100kV	680	800
	120kV	570	835
	140kV	490	715
Hi-Res	80kV	620	700
	100kV	500	750
	120kV	420	625
	140kV	360	540

Permissible mA values for single 10-second exposure from cold

Scan Mode	kV	Small Focal Spot mA	Large Focal Spot mA
Normal	80kV	620	700
	100kV	650	800
	120kV	540	755
	140kV	460	650
Hi-Res	80kV	610	700
	100kV	490	710
	120kV	405	590
	140kV	350	505

Serial Load Rating

Permissible mA values for serial 5-second exposures (10 min IPD)

Scan Mode	kV	Small Focal Spot mA	Large Focal Spot mA
Normal	80kV	620	700
	100kV	610	800
	120kV	505	705
	140kV	435	605
Hi-Res	80kV	575	700
	100kV	460	670
	120kV	380	555
	140kV	325	475

Permissible mA values for serial 10-second exposures (10 min IPD)

Scan Mode	kV	Small Focal Spot mA	Large Focal Spot mA
Normal	80kV	620	700
	100kV	550	760
	120kV	455	630
	140kV	390	540
Hi-Res	80kV	535	700
	100kV	430	600
	120kV	355	500
	140kV	305	425

© 2014 General Electric Company.

GE and GE Monogram are trademarks of the General Electric Company.

The Performix brand is a registered trademark of GE Healthcare.

General Electric Company, doing business as GE Healthcare.

General Electric Company reserves the right to make changes in specifications and features shown herein or discontinue the product described at any time without notice or obligation. Contact your GE Representative for the most current information.

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